

# The CLYDE Range

## Compact Superior 2- 'two utensils at a time' Washer Disinfecter CS2

Utensil loading configuration



The Compact Superior 2 is specifically designed for multi storey care homes where code of practice requires a sluice room on each floor.


This CS2 medium sized top loading machine washes and disinfects a variety of utensils two at a time. For care homes committed to disposable products it is an ideal, versatile washer disinfecter for the carriers and reusable utensils to complete the infection control barrier.

All Stanbridge washer/disinfecters can include the DEOSCALE water treatment system to control lime scale build-up and reduce maintenance.

**2 off** Highdome version only




Slipper

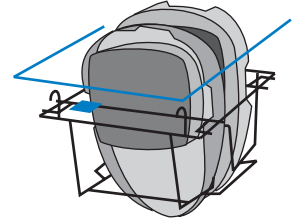

**2 off** Highdome version only




Commode Insert pan

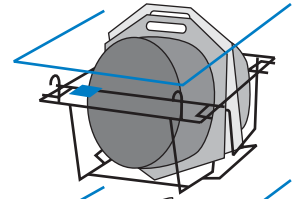

**2 off**




In Bed pan

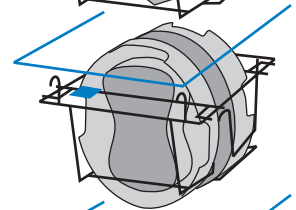

**2 off**




Commode pot and lid

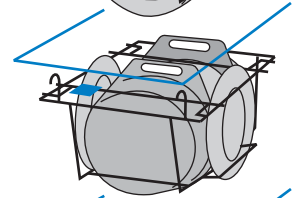

**2 off**




Support Carrier for cardboard

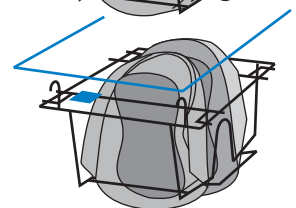


**1 off**




Urine bottle

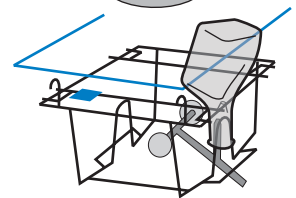

**2 off**

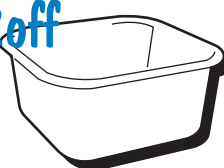


Basin (Square)

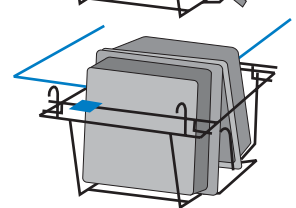

**1 off**



Grey Pot

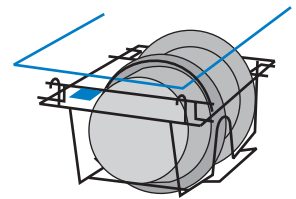

## Stanbridge Infection Control

Stanbridge Ltd, Cray Valley Road, Orpington, Kent BR5 2UB

Tel: 01689 806500 Fax: 01689 806501 email: [sales@stanbridge.co.uk](mailto:sales@stanbridge.co.uk) [www.stanbridge.co.uk](http://www.stanbridge.co.uk)

Stanbridge reserves the right to make design changes without notification

# Compact Superior 2 - CS2

Size	Installation	Shipping
Height	910(1030)*mm (36")	1010(1130)*mm(40")
Width	435mm (18")	535mm (22")
Depth	600mm (24")	760mm (30")

\*Highdome version

**Weights** 85Kg Tanks full 77Kg

## Water Feed

Supply Cold  
Connection 1/2" BSP male parallel  
Minimum Flow 0.19lt./sec (2.65gal/min)  
Temperature Cold Ambient  
Hot 60-65 deg C  
Consumption 23.5lt(5.25gall) per normal cycle

**Overflow** 1 x 19mm (3/4")bore.  
End must be visible (UK requirement)  
Pipework by client

**Waste** 82mm (3") 'P' or 'S' trap

**Electrical** 230V 50Hz 13A single phase  
Fuse - mains 13A - control 5A  
Steam generator 3.0 kW  
Pump 0.37kW  
Consumption 0.21kWH per cycle av.

## Washing Cycles

1. Normal wash  
Wash - disinfection at 90 deg. C for  
70 sec - rinse to cool utensils
2. Super wash  
Pre flush - wash - disinfection at 90  
deg. for 70 sec - rinse to cool utensils

**Sound Level** 73 dB

## Instrumentation Panel

Normal wash start button, super wash  
start button - digital temperature  
indicator - override key switch - lights  
for wash, steam and cycle complete.

Stanbridge utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings. Stanbridge Ltd is approved under BS 5750 for quality assurance.

WRAS Approval No. 0310064  
Internal break tank has a type 'A' air gap supply

