

Slimline










Front loading Washer Disinfector

Loading configuration



The Slimline front loading traditional bed pan/bottle washer-disinfector is a narrow machine with a small footprint, ideally suited where space in the sluice room is at a premium. Its multi utensil use suits high bed occupancy and high dependency wards.

All Stanbridge washer/disinfectors can include the DEOSCALE water treatment system to control lime scale build-up and reduce maintenance.

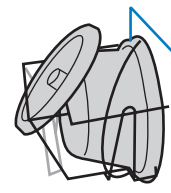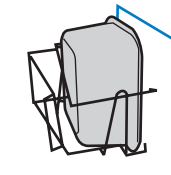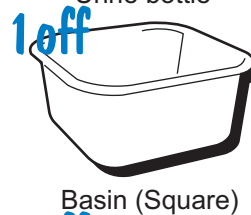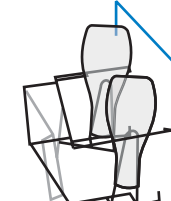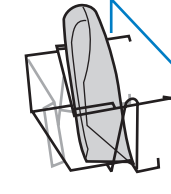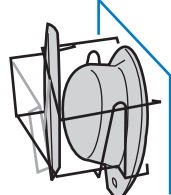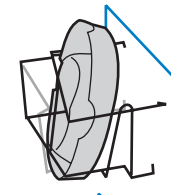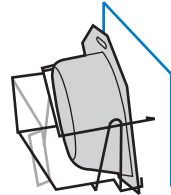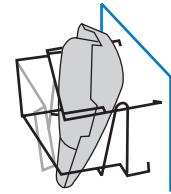

Stanbridge Infection Control

Stanbridge Ltd, Cray Valley Road, Orpington, Kent BR5 2UB

Tel: 01689 806500 Fax: 01689 806501 email: sales@stanbridge.co.uk www.stanbridge.co.uk

Stanbridge reserves the right to make design changes without notification

Slimline

Size	Installation	Shipping
Height	1500mm (60")	1600mm (63")
Width	550mm (22")	620 mm (25")
Depth	490mm (20")	590mm (24")

Weights	95Kg	105Kg
----------------	------	-------

Water Feed

Supply	Hot and cold	
Connection	1/2" BSP male parallel	
Minimum Flow	0.19lt./sec (2.65gal/min)	
Temperature	Cold	Ambient
	Hot	60-65 deg C
Consumption	23.5lt(5.25gall) per normal cycle	

Overflow	1 x 19mm (3/4")bore. End must be visible (UK requirement) Pipework by client
-----------------	--

Waste	82mm (3") 'P' or 'S' trap
--------------	---------------------------

Electrical	230V 50Hz 13A single phase Fuse - mains 13A - control 5A Steam generator 3.0 kW Pump 0.37kW Consumption 0.21kWH per cycle av.
-------------------	---

Washing Cycles

1. Normal wash
Wash - disinfection at 85 deg. C for
70 sec - rinse to cool utensils
2. Super wash
Pre flush - wash - disinfection at 85
deg. for 70 sec - rinse to cool utensils

Sound Level	73 dB
--------------------	-------

Instrumentation Panel

Normal wash start button, super wash start button - digital temperature indicator - override key switch - lights for hot wash, cold wash, steam and cycle complete.

Stanbridge utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings. Stanbridge Ltd is approved under BS 5750 for quality assurance.

WRAS Approval No. 0310064
Internal break tank has a type 'A' air gap supply

