

Urine Bottle Washer

Nine-at-a-time

Urology wards process a large number of urine bottles daily, up to 150 items with peak loads.

The Compact CS29U top loading washer disinfector has a 9 station high bottle capacity to match the work load and reduces wait times in urology departments. It also improves the control of cross infection.

Jugs can be part of the mixed loads including the large 2 litre size which is favoured to avoid splash when transferring contents from bags. Without proper facility to disinfect large jugs, cross infection barriers will be breached.

Blood spots are removed by a unique design feature which provides separate water power at the centre of every utensil. This ensures complete impact coverage.

Substantial savings are achieved with multi-utensil washing against the disposable pulp system where costs are considerable in high use urology wards. Throw away cardboard costs drain ward cash flows.

An accurate water dosing system aids cleaning, reduces utensil discolouration, deodorises and controls lime scale.



Stanbridge Infection Control

Stanbridge Ltd, Cray Valley Road, Orpington, Kent BR5 2UB

Tel: 01689 806500 Fax: 01689 806501 email: sales@stanbridge.co.uk www.stanbridge.co.uk

Stanbridge reserves the right to make design changes without notification

Urine Bottle Washer

Size	Installation	Shipping
Height	910mm (36")	1010mm(40")
Width	435mm (18")	535mm (22")
Depth	600mm (24")	760mm (30")

Weights 85Kg Tanks full 77Kg

Water Feed

Supply Cold
Connection 1/2" BSP male parallel
Minimum Flow 0.19lt./sec (2.65gal/min)
Temperature Cold Ambient
Hot 60-65 deg C
Consumption 23.5lt(5.25gall) per normal cycle

Overflow 1 x 19mm (3/4")bore.
End must be visible (UK requirement)
Pipework by client

Waste 82mm (3") 'P' or 'S' trap

Electrical 230V 50Hz 13A single phase
Fuse - mains 13A - control 5A
Steam generator 3.0 kW
Pump 0.37kW
Consumption 0.21kWH per cycle av.

Washing Cycles

1. Normal wash
Wash - disinfection at 90 deg. C for
70 sec - rinse to cool utensils
2. Super wash
Pre flush - wash - disinfection at 90
deg. for 70 sec - rinse to cool utensils

Sound Level 73 dB

Instrumentation Panel

Normal wash start button, super wash
start button - digital temperature
indicator - override key switch - lights
for wash, steam and cycle complete.

Stanbridge utensil washer disinfectors are manufactured in compliance with the recommendations of British Standard 2745 and to meet the detail requirements of inspectors of the Water Supply Undertakings. Stanbridge Ltd is approved under BS 5750 for quality assurance.

WRAS Approval No. 0310064
Internal break tank has a type 'A' air gap supply



Stanbridge Infection Control